EN This unique rinsing solution is formulated with OcuPure® preservative to be gentle in your eyes.

LENs PLUS® OcuPure® Saline is the natural choice for rinsing your lenses after disinfection and is also ideal for dissolving protein remover tablets.

Instructions for Use: Always wash and dry your hands thoroughly before handling your contact lenses. After cleaning and disinfecting, rinse your lenses thoroughly with LENs PLUS® OcuPure® Saline prior to insertion. Follow the manufacturer's instructions to use protein remover tablets correctly.

Precautions:

- All instructions must be followed. Failure to do so may result in serious eye infection or injury.
- Do not use if tamper-evident seal is broken or missing
- Do not use if allergic to any of the ingredients
- If eye irritation occurs, discontinue lens wear and consult your eye care practitioner
- Always use fresh solution
- Do not reuse the expiry date
- Close the bottle tightly immediately after use
- To avoid contamination, do not allow the bottle tip to touch any surface
- Discard any remaining solution within 90 days after first opening the bottle
- Store between 5°C and 25°C
- Keep out of the reach of children
- Do not rinse your lenses or lens case with tap water
- Do not swallow the solution

NOT FOR CONTACT LENS DISINFECTION

Contents:

Sodium chloride, boric acid, OcuPure® preservative (sodium chlorate) - 0.005% and purified water.

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CE 0123 **STERILE A**

MADE IN INDIA

Block B, Liffy Valley Office Campus
Quarryvale Co. Dubhail, Ireland

LENs PLUS, LENs PLUS OcuPure and OcuPure are trademarks of Johnson & Johnson Surgical Vision, Inc.
© Johnson & Johnson Surgical Vision, Inc. 2018

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AM01504

DC • Labeling & Packaging

Platform: MAC - Adobe Illustrator CS4 Last Update: 24/04/2018 - 16:30 v1 <i>Note: This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards. Please click on a specific colour</i>		- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMEARING/BROKEN TYPE IS NOT ACCEPTABLE.	